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# Ceraderm Betaglucan

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Denna bipacksedel reviderades senast den 5.2.2021

CE 0537

Package leaflet  
**Ceraderm Betaglucan**

**Ceraderm Betaglucan is an occlusive ointment containing oat beta glucan. It is indicated in the treatment of superficial burns and damaged, irritated and dry skin.**

Read this leaflet carefully before you start using this product because it contains important information for you.

**1. What Ceraderm Betaglucan is and what it is used for**

Ceraderm Betaglucan is a fatty occlusive ointment protecting the skin. It is indicated in the treatment of superficial burns and damaged, irritated and dry skin. The ointment can also be used for the treatment of other skin problems, such as psoriasis or atopic skin.

Ceraderm Betaglucan

- moistens the skin and protects it from external irritants
- prevents heat loss
- controls itching, irritation and pain of the skin
- contains oat beta glucan which binds moisture to the skin and forms a protective layer
- contains no preservatives or fragrances

Ceraderm Betaglucan is suitable for both children and adults. It can be applied to the whole body, including the face.

**2. What you need to know before you use Ceraderm Betaglucan**

Do not use Ceraderm Betaglucan if you are allergic to any of the ingredients of this product (listed in section 6). If you have received separate instructions for use of the product from your doctor or healthcare personnel, always follow such instructions.

**Warnings and precautions**

Ceraderm Betaglucan is unsuitable for the treatment of preterm babies. If the ointment irritates the skin or causes skin symptoms, stop using it and consult your doctor or the person treating you.

The ointment is not harmful if swallowed. Avoid contact of the ointment with eyes. In case of accidental eye contact rinse your eyes with plenty of water. Contact a doctor if necessary.

**3. How to use Ceraderm Betaglucan**

Apply Ceraderm Betaglucan to clean skin. Apply the ointment sparingly to the affected skin area once or twice daily or as instructed. The ointment is very economical to use.

Use clean hands or a spatula, for instance, to apply the ointment. Do not allow the ointment tube to come in contact with the affected skin because impurities might get into the tube.

**Treatment of superficial burns**

Ceraderm Betaglucan can be used to treat superficial burns. Rinse the skin until clean and carefully dab it dry. Apply the ointment to the affected skin area. Protect the area with a bandage, if necessary. Re-apply the ointment and change the bandage, if necessary. Always follow the instructions given by your doctor or by the healthcare personnel.

If the burn does not start healing or gets worse (e.g. increased pain, swelling or odour), stop using the ointment and consult a doctor or healthcare professional.

**Treatment of bruises and cuts**

Ceraderm Betaglucan can be used to treat acute bruises and superficial cuts (e.g. grazes, fissures, sores). Clean

the skin with water or with an antiseptic skin cleanser, if necessary. Dab the skin dry. Apply the ointment to the affected skin area. Cover the skin with a bandage, if necessary. Re-apply the ointment and change the bandage, if necessary. Always follow the instructions given by your doctor or by the healthcare personnel.

If the cut or bruise does not start healing or gets worse (e.g. increased pain, swelling or odour), stop using the ointment and consult a doctor or healthcare professional.

N.B.: Ceraderm Betaglucan is not indicated in the treatment of chronic wounds.

**Other considerations related to use**

Ceraderm Betaglucan is only intended for topical use on the skin.

Do not use the ointment if the tube is damaged or after the expiry date. The expiry month is marked on the crimp of the tube. Please note that the ointment should be used within 12 months from opening the tube.

**4. Possible side effects**

Ceraderm Betaglucan has no known adverse effects. However, if you suspect that the ointment has caused you inappropriate symptoms or reactions, please tell your doctor, pharmacist or nurse.

**5. How to store Ceraderm Betaglucan**

Store the ointment at room temperature, below 25°C. It will keep stable for 12 months after opening.

Keep the product out of the sight and reach of children. Do not use the ointment if the tube is damaged or after the expiry date. The expiry month is marked on the crimp of the tube.

The empty tube can be discarded with ordinary household waste. Ask your pharmacist how to dispose of unused or partly used tubes.

**6. Contents of the pack and other information**

**What Ceraderm Betaglucan contains**

The active substances are petrolatum, jojoba oil (*Simmondsia Chinensis*) and oat beta glucan. The other ingredients are isododecane, isopropyl myristate, microcrystalline wax and silicon dioxide (silica).

**What Ceraderm Betaglucan looks like and contents of the pack**

Light, fatty ointment with mild oat scent. 30 g, 100 g and 150 g plastic tubes

Marketer/Distributor  
Bitelab Oy and Decem Pharma Oy  
Pakkalankuja 6, 01510 Vantaa, Finland  
Tel: +358 10 8305800  
info@bitelab.fi

Manufacturer  
Galena Pharma Oy  
Sammonkatu 10, 70500 Kuopio, Finland  
Tel: +358 17 2881250  
galena@galena.fi

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